

Fact plus[®]

Pregnancy Test
Prueba De Embarazo

PLEASE READ INSTRUCTIONS CAREFULLY BEFORE USE

Over 99% Accurate*
from the day of your expected period.



READY TO TEST?

Use Fact Plus pregnancy test for results 5 days sooner than missed period.

BEFORE YOU BEGIN:

- Read instructions **COMPLETELY** and have a watch, clock, or timer available.
- Check expiration date on box. Do not use if expired.

HOW TO USE:

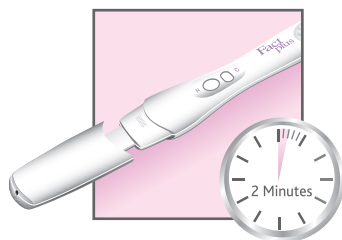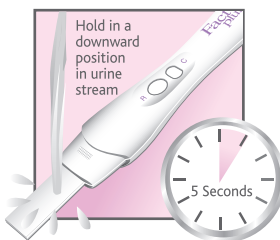

1. Open the protective foil pouch **IMMEDIATELY** before use.
2. Remove the test stick from the foil package. Take off the cap.
3. Hold the test stick by the thumb grip. **Point the Absorbent Tip downward** in urine stream for **5 seconds ONLY. IMPORTANT: ONLY 5 SECONDS.** Too much or too little urine on Absorbent Tip may cause an invalid result. (See Invalid Result section below.)
4. If you prefer, urinate into a clean, dry cup and dip all of the Absorbent Tip into the cup for **20 seconds.**
5. Replace cap and lay on flat surface with "Fact Plus" facing up.

HOW TO READ RESULTS:

You can read your results after **2 minutes**. A blue line will appear in the Control (C) Window indicating the test has worked. A "-" in the Result (R) Window indicates a negative result. A "+" in the Result (R) Window indicates a positive result.

- **DO NOT** read result after 10 minutes.
- If after 10 minutes **NO** line appears in the Control (C) Window, the result is invalid. Repeat with new Fact Plus test. (See Question # 7)

RESULT AREA

Positive Result
One blue line in Control (C) Window **AND** a "+" in Result (R) Window.



Negative Result
One blue line in Control (C) Window **AND** a "-" in Result (R) Window.



Invalid Result
No line appears in Control (C) Window.



It does not matter if one of the lines that make up the "+" symbol is fainter or darker than the other.





Over 99% Accurate*
from the day of your expected period.

Questions & Answers (Q&A)

1. How does Fact Plus work?

Fact Plus detects hCG (human Chorionic Gonadotropin), a hormone present in urine once you become pregnant. A "+" (positive) result indicates that hCG has been detected.

2. When can I test?

Fact Plus can be used as early as 4 days before your expected period. If you test before your expected period and get a "-" (negative) result, there is still a chance that you may be pregnant. See the chart below with clinical results from early testing with Fact Plus.

Number of Days Before	% of women getting a "+" (positive) result with Fact Plus.
-4 Days	53%
-3 Days	74%
-2 Days	84%
-1 Day	87%

3. Do I have to use first morning urine?

Your first morning urine usually contains a higher concentration of hCG. If testing early (before your expected period) it is recommended you use your first morning urine. Otherwise, you can test at any time of day. Please note, drinking large amounts of fluid before testing may dilute the hCG in your urine and cause an inaccurate result.

4. I have irregular periods so I am not sure when my next period is due. How do I know I am testing at the right time?

With irregular periods you should test according to your longest cycle in recent months. If you are uncertain when your period is due, we recommend testing at least 19 days from the day you think you became pregnant. If the result is negative you should test again in 3 days because you may have tested too early.

5. How soon can I read the result?

You can read your result after 2 minutes as long as a blue line has appeared in the Control (C) Window. A "+" (positive) result may appear as early as 1 minute. Disregard all results after 10 minutes.

6. Are there any factors that can affect the test result?

- Always read the manufacturer's instructions for any medication you are taking before the test.
- Fertility drugs containing hCG (such as Pregnyl® and Profasi®)** can affect the result. Testing with Fact Plus too soon after administration may give a false "+" (positive) result.

Questions?

Call toll-free **1-800-986-8598**

Monday-Friday, 8:30 a.m.-5:00 p.m. Eastern Time.

Our professional staff is available to assist you.

Visit our website at www.factplus.com
(Instructions in both English and Spanish).

- Other fertility therapies (such as Clomid®)***, painkillers and the contraceptive pill should not affect the result.
- If you have recently stopped taking the contraceptive pill or are using fertility therapies like Clomid®***, your periods may be irregular, leading you to test too soon.
- If you have recently been pregnant (even if not carried to full term) you may get a false "+" (positive) result. In addition, ovarian cysts, ectopic pregnancy, menopause and some very rare medical conditions can give misleading results.
- If you do get unexpected results you should discuss them with your doctor.

7. What might affect the accuracy of the result?

- If testing before your expected period, the test accuracy will be lower. (See chart in Question 2)
- Following test procedure incorrectly, including specified sample time, not keeping the Absorbent Tip pointing down, and expiration of the test are causes of an Invalid Result.
- If you do get unexpected results, you should check with your health care professional.

8. My test says I'm pregnant, what should I do?

You should see your health care professional who can advise you on what steps you should take next.

9. My test says I'm not pregnant, but I think I am. What should I do?

A "-" (negative) result means no hCG has been detected. Wait 3 days and test again using a new Fact Plus pregnancy test. If you still get a "-" (negative) result and your period has not started, you should see your health care professional.

*Over 99% accurate at detecting pregnancy hormone. However, some pregnant women may not have detectable amounts of pregnancy hormone in the urine on the first day of the missed period or may have miscalculated the first day of their period.

**Pregnyl® is a registered trademark of Organon USA, Inc. Profasi® is a registered trademark of Ares Trading S.A.

***Clomid® is a registered trademark of Merrell Pharmaceuticals, Inc.

Questions? Call toll free 1-800-986-8598 Monday-Friday 8:30 a.m. -5:00 p.m Eastern Time.

Our professional staff is available to assist you.

Visit our website at www.factplus.com

Do not use a test that is past expiration date.

Fact Plus is a pregnancy test for home use. Store at 36°F to 86°F (2°C to 30°C). Do not freeze. For in vitro diagnostic use. Not for internal use. Keep out of reach of children. Do not use if the foil wrapper is damaged.

Distributed by Procter & Gamble, Cincinnati, OH 45202.

Fact Plus is a registered trademark of SPD Swiss Precision Diagnostic GmbH.

© 2008 US CD LLC. All rights reserved.

P/N 000264-10 4/08